

Bite jumping screw*



Bite jumping screw*
for the modified Twin Block appliance

New!



No more acrylic work needed for progressive advancement of the twin block appliance during treatment

No dismantling of the screw, no spacers needed



CE 0297

Easy and safe chairside adjustment by just activating the screw

Patient friendly progressive advancement improves compliance

FORESTADENT[®]

Bite jumping screw*

CE 0297

Progressive Overjet Reduction for the modified Twin Block appliance




The Twin Block appliance by Dr. William Clark is one of the most popular functional appliances for Class II, division 1 malocclusion treatment.

Standard Twin Block appliances however can only be reactivated by a time consuming procedure.



This modification allows a progressive advancement of the bite, providing a greater orthopedic effect with less incisor tilting in Class II, division 1 cases. Progressive advancement of the mandible helps to reduce tension in the cranio-mandibular muscles, therefore improving patient comfort and compliance.

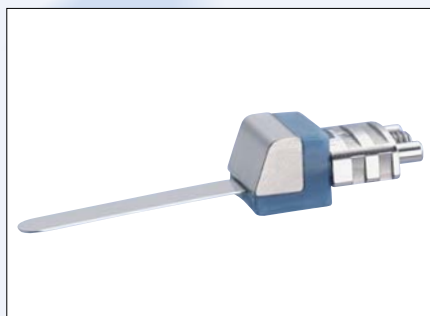

Easy and safe chairside reactivation of as much as 6 mm can be achieved by incorporating this solid stainless steel screw into the appliance. Chairside reactivations provide minimal inconvenience to staff and patient. This modified Twin Block has great advantages for patients with severe overjets or limited ability to advance their mandible for the construction bite.

Art.No. 199-2622 pack of 2 screws CE 0297

FORESTADENT®

Bernhard Förster GmbH · Westliche Karl-Friedrich-Straße 151 · 75172 Pforzheim · Germany
Phone + 49 7231 459-0 · Fax + 49 7231 459-102 · info@forestadent.com · www.forestadent.com
Forestadent USA · 2301 Weldon Parkway · St. Louis, MO 63146 · TOLL FREE 1-800-721-4940
Fax 314-878-7604 · info@forestadentusa.com · Internet: http://www.forestadentusa.com
Forestadent Ltd. · 21 Carters Lane/Kiln Farm · Milton Keynes MK11 3HL · Great Britain
TOLL FREE Phone 0 800 056 39 01 · Fax + 44 1 908 560 611 · info@forestadent.co.uk